## FELICITACIONES!

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## CONGRATULATIONS!

You have joined the group of users of the KMD technology.

In order to ensure a prolonged use and optimum performance of your instruments, we advise you to read these instructions carefully.

En

### CHARACTERISTICS

- Long life span.
- Mounted on four ball bearings.
- Does not diminish the power of the micromotor.
- Up to 25.000 r.p.m.
- External spray.

### TECHNICAL SPECIFICATIONS

**Upper and lower shafts mounted on ball bearings  
For 2.35 mm standard drills  
Drill secured through button  
Transmission 1:1  
Net Weight: 40 grs.**

### CONNECTION AND WORKING

Check that your micromotor has an Intra type coupling. Then push the button, place the drill turning slowly until that the same sockets, as indicated in the fig. 1 and to extract fig.2. For the refrigeration it is provided with a hose to connect it to the micromotor externally.

### DESINFECTATION AND STERILIZATION

Disconnect the contra angle from the micromotor. Clean the same exteriorly with alcohol or another desinfectant solution. DO NOT SUBMERGE IT IN DESINFECTANT SOLUTION. Lubricate the contra angle with KMD spray oil injected during two seconds fig. 4. Wrap it in e paper. Sterilize in autoclave up to 135°C (275°F) during one cycle. Withdraw the instrument and connect it to the micromotor, running it for a few seconds, so the excess oil will be eliminated.

### MAINTENANCE

Lubricate after every working day with our KMD Spray Oil or another similar product apt to bearings contra angle. The correct mode is indicated in the fig.4.

### TROUBLESHOOTING : BEFORE SENDING YOUR INSTRUMENT TO A QUALIFIED SERVICE PERSON, FIRST VERIFY

PROBLEM	CAUSE	CORRECTION
It does not fit into the micromotor	-The micromotor has another system	-Check with your dealer -Press until it fits (see fig. 3)
The drill does not rotate	-It is not properly connected -The drill is not properly placed	-Check if it is the correct drill then proceed according to instructions. (see fig. 1)
The drill turns intermittently Water does not flow	-The drill is not properly placed  -Water hose obstructed -Spray tube obstructed	-Repeat the operation according to fig. 1  -Check that water comes out of the micromotor -Proceed according to instructions