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Manual de Instrucciones

CAVITADOR GEMA

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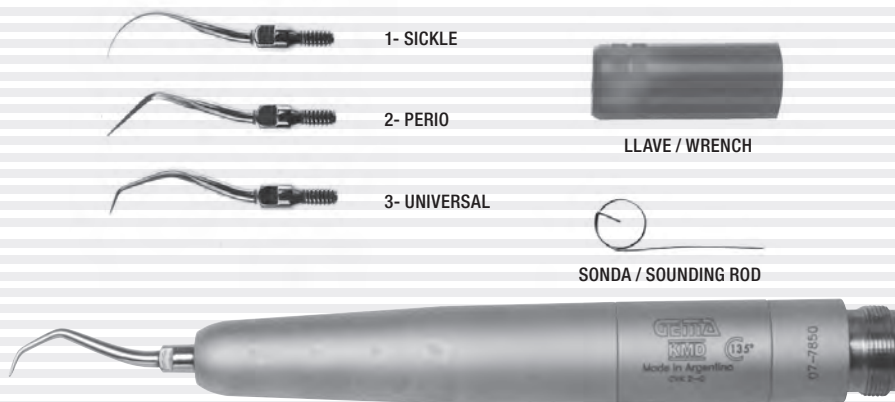
En

Congratulations!

You have joined the group of users of the KMD technology. In order to ensure a prolonged use and optimum performance of your instruments, we advise you to read these instructions carefully.

Fig. A

Technical specifications:

Maximum air pressure:	2,10 kg/cm ² (30 P.S.I.)
Net weight:	55 gr.
Frequency:	6000 Hz aprox.

CHARACTERISTICS

- Elliptical movement of the point, very efficient in removing calculus.
- Its stylized design, the anatomic handle and its low weight reduce fatigue in prolonged treatments.
- Spray in refrigeration gives the surface to be treated a levelled and constant temperature.
- Easy installation.
- Autoclavable 135 °C.

INSTALLATION AND WORKING

Verify that the coupling of equipment is the same one which Air Scaler Gema has got, then control that the air and water get out through the connector, twist the nut of coupling clockwise and fasten.

Then choose the inserted and screw manually clockwise (Fig 1) and adjust with the wrench provided.

Fig. 1

Fig. 2

Fig. 3

The extractor key is designed to protect the points at the moment of interchanging and/or when the instrument is in the base of the equipment.

For a correct working of the air scaler Gema the air pressure has to be from 2,10 kg/cm² or 30 p.s.i. Once regulated the air pressure, make working the air scaler without water, slowly add the step of water until it forms a fine drizzle in the tip. It is indispensable to supply dry and clean compressed air.

DESINFECTATION AND STERILIZATION:

Disconnect the air scaler from the equipment. Clean it's external parts with alcohol or another disinfectant solution.

DO NOT SUBMERGE IT IN DESINFECTANT SOLUTION.

Lubricate the air scaler with KMD spray oil injected during two seconds. Sterilize in autoclave up to 135°C (275°F) during one cycle.

Withdraw the instrument and connect it to the line of air, operating the same during some seconds, so will be eliminated the oil exceeding.

MAINTENANCE

Lubricate after every working day with our KMD Spray Oil or another similar product apt to air scaler.

We recommend using KMD Spray Oil, which must be injected for two seconds, following the indication of the figure A.

Lubricate: To uncover the passages of water: In the case of the point: inject air with the syringe through the water circuit; if it is not enough with it, use the sound supplied. (fig. 2)

In the case of the instrument: inject air with the syringe through the circuit of water. (fig. 3)

❏ Fault*	🔍 Causa	🔧 Corrección
No vibration	1-No air arrive.	1-Check air pressure of compressor and unit.
Low vibration	1-Very little air pressure. 2-Loose tip. 3-Alteration of the tip form.	1-Check pressure of unit. 2-Adjust securely with the wrench. 3-Change it for a new one.
No water supply	1-Water line is shut off. 2-Tip or instrument close up.	1-Check water go out from the connection .Check the unit. 2-Go on like figure 2 and figure 3.
Water leaks at connector.	1-Loose connector nut 2-Connection join faulty.	1-Adjust securely. 2-Change it for a new one.
Broken insert	Excess of pressure	Adjust working pressure of the equipment
Interrupted sound.	1-Loose tip.	1-Adjust securely with the wrench.

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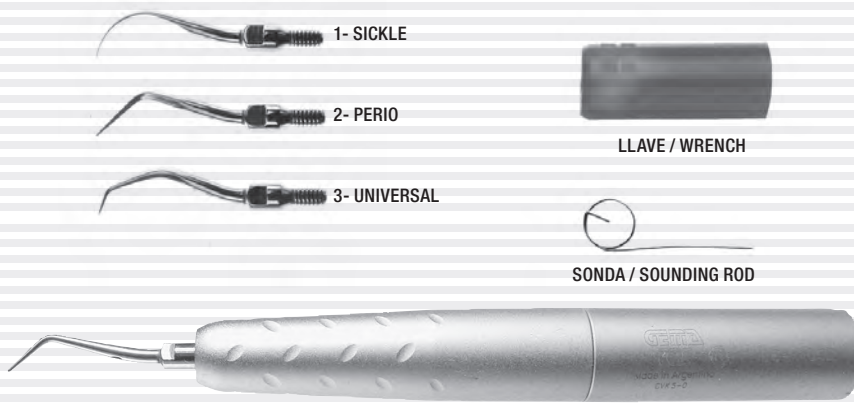
Operating Instructions

SCALER SONIC GEMA MULTIFLEX **En**
 Congratulations!
 You have joined the group of users of the KMD technology.
 In orders to ensure a prolonged use and optimum performance of your instruments, we advise you to read these instructions carefully.

- CHARACTERISTICS**
- Elliptical movement of the point, very efficient in removing calculus.
 - Its stylized design, the anatomic handle and its low weight reduce fatigue in prolonged treatments.
 - Spray in refrigeration gives the surface to be treated a levelled and constant temperature.
 - Autoclavable 135°C.
 - Easy installation.

The extractor key is designed to protect the points at the moment of interchanging and/or when the instrument is in the base of the equipment.
 For a correct working of the air scaler Gema the air pressure has to be from 2,80 kg/cm2 or 40 p.s.i. Once regulated the air pressure, make working the air scaler without water, slowly add the step of water until it forms a fine drizzle in the tip.
 It is indispensable to supply dry and clean compressed air.

INSTALLATION AND WORKING
 Verify that the coupling of equipment is the same one which Air Scaler Gema has got, then control that the air and water get out through the connector, twist the nut of coupling clockwise and fasten.
 Then choose the inserted and screw manually (Fig 1) clockwise and adjust with the wrench provided.

DESINFECTATION AND STERILIZATION:
 Disconnect the air scaler from the equipment. Clean the same exteriorly with alcohol or another disinfectant solution.
DO NOT SUBMERGE IT IN DESINFECTANT SOLUTION.
 Lubricate the air scaler with KMD spray oil injected during two seconds.
 Sterilize in autoclave up to 135°C (275°F) during one cycle.
 Withdraw the instrument and connect it to the line of air, operating the same during some seconds, so will be eliminate the oil exceeding.

Fig. A

MULTIFLEX

Technical specifications:	
Maximum air pressure:	2,80 kg/cm ² (30 P.S.I.)
Net weight:	55 gr.
Frequency:	6000 Hz aprox.
Coupling:	Multiflex

Fig. 1

Fig. 2

Fig. 3

MAINTENANCE
 Lubricate after every working day with our KMD Spray Oil or another similar product apt to air scaler.
 We recommend using KMD Spray Oil, which must be injected for two seconds, following the indication of the figure A.
 Lubricate: To uncover the passages of water: In the case of the point: inject air with the syringe through the water circuit; if it is not enough with it, use the sound supplied. (fig. 2)
 In the case of the instrument: inject air with the syringe through the circuit of water. (fig. 3)

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Fault*	Causa	Corrección
No vibration	1-No air arrive.	1-Check air pressure of compressor and unit.
Low vibration	1-Very little air pressure. 2-Loose tip. 3-Alteration of the tip form.	1-Check pressure of unit. 2-Adjust securely with the wrench. 3-Change it for a new one.
No water supply	1-Water line is shut off. 2-Tip or instrument close up.	1-Check water go out from the connection .Check the unit. 2-Go on like figure 2 and figure 3.
Water leaks at connector.	1-Loose connector nut 2-Connection join faulty.	1-Adjust securely. 2-Change it for a new one.
Broken insert	Excess of pressure	Adjust working pressure of the equipment
Interrupted sound.	1-Loose tip.	1-Adjust securely with the wrench.

*Troubleshooting. Before sending your instrument to a qualifier service person, first verify:

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* Multiflex is a KATENBACH UND VOIGT GmbH trademark.