# KMD

WWW.KMDARGENTINA.COM.AR

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## Operating Instructions

Congratulations!

You have joined the group of users of the KMD technology.

In order to ensure a prolonged use and optimum performance of your instruments, we advise you to read these instructions carefully

En

### CHARACTERISTICS

- Constant number of revolutions.
- Suitability for any trademark of contra angles or rights handpieces using INTRA system.
- Adjustment of the number of revolutions in motion and in reverse gear.
- 360° turn of the contra angle or right handpiece with out effort.

### MAINTENANCE

Lubricate after every working day with our KMD Spray Oil or another similar product apt to micromotor. The correct mode is indicated in Fig.3. It is indispensable to supply dry clean compressed air.

### CONNECTION AND WORKING

Verify that the connection of the equipment is the same as that of the micromotor, then check that the air and water get out through the connection, screw in the adjusting nut clockwise and adjust. Once these instructions have been carried out, your micromotor will be installed.

For the adjustment in motion and in reverse turn the command ring, like fig. 1. For a correct working of the micromotor the minimum air pressure have to be from 2,10 kg/cm2 or 30 p.s.i. The refrigeration system is external.

### DESINFECTATION AND STERILIZATION

Disconnect the micromotor from the equipment. Clean the same exteriorly with alcohol or another desinfectant solution. **DO NOT SUBMERGE IT IN DESINFECTANT SOLUTION.** Lubricate the micromotor with KMD spray oil injected during two seconds. Wrap it in paper.

Sterilize in autoclave up to 135°C (275°F) during one cycle. Withdraw the instrument and connect it to the line of air, operating the same during some seconds, so will be eliminate the oil exceeding.

### BORDEN 2 BORDEN 3 MIDWETS 4

## TECHNICAL SPECIFICATIONS

Maximum air pressure	2,10 kg/cm2 (30 P.S.I.)
Adjustable speed	5000 a 20000 r.p.m.
Couplinge	Suitable for INTRA instruments
Net weight	75grs.

**Troubleshooting.** Before sending your instrument to a qualified service person, first verify:

ERROR	CAUSE	CORRECTION
The micromotor does not start	1- The control ring is set halfway between motion and reverse.	1- Move the control ring towards motion or reverse as required.
Insufficient speed or torque	1- Lack of air or air pressure. 2- Lack of lubrication.	1-Inspect and adjust air system. 2-Lubricate according to instructions.
Noisy or defective working of the micromotor.	1- Lack of air. 2- Lack of lubrication.	1-Inspect the hose and the connections. 2-Lubricate according to instructions.